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Please note:

Our Quick Start Guide does not replace reading and observation of the detailed operating manual.

You will find there more details as well as information about changing the program, charging the control unit and cleaning.

1

1a. Put the sealing ring and the inhalation mask over the horses muzzle and secure the fastening strap by plugging the two quick-release fasteners into the counterpart.

2

2a. Slide the control unit along the slide rails until it snaps in.

2b. Insert a filter membrane and slide the filter basket over the two bolts of the mask.

3

3a. Fill the prescribed substance into the mesh nebulization unit.
Please read the detailed operating manual!

4

4a. Connect the mesh nebulization unit by leading the connection cable to one of the two sockets of the control unit.

4b. Slide the protection cover along the side rails until it clicks into place.

5

5a. Turn on the control unit by holding down the power button through the opening in the protection cover.

5b. The control unit switches off automatically as soon as there is no residual quantity remaining.