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
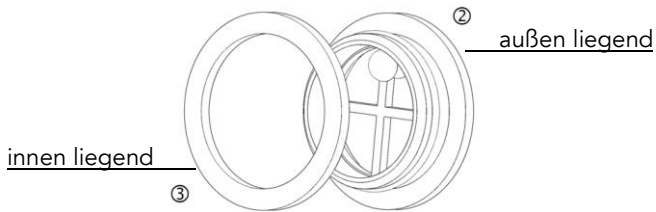
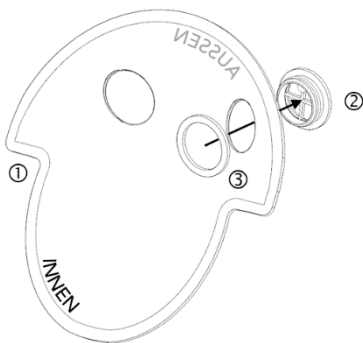
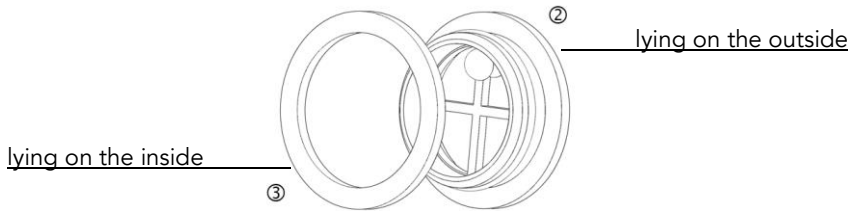
The article SaHoMa®-II filter membrane with valve set (art.-no.: HM-213) contains five filter membranes with holes and a valve set, as shown in the picture below, consisting of two valve seats ② and two valve rings ③:

The valve set is designed as a reusable plug-in system and is attached to one SaHoMa®-II filter membrane by the manufacturer. It should be removed after consumption of the SaHoMa®-II filter membrane and then inserted into a new filter membrane as described below:

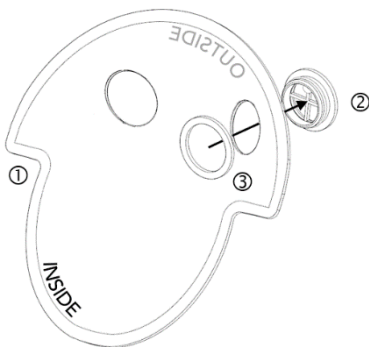R  
E  
L  
E  
A  
S  
E


*Before releasing the valves, please have a look at the SaHoMa®-II filter membrane and memorize how the valve set sits in the filter membrane.*

### **Releasing the valves by pulling off the valve ring**

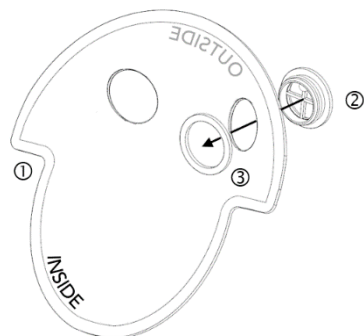To release the valve, take the valve ring ③ sitting on the inside of the filter membrane ① and release it by pulling it slightly off the valve seat ②.

I  
N  
S  
E  
R  
T

### **Inserting the valves by attaching the valve ring**

To insert the valves, put the valve seat ② on the outside of the filter membrane ① through the provided hole and secure it by pressing the valve ring ③ sitting on the inside of the filter membrane ① onto the valve seat ② all the way to the stop.

Together with the last used SaHoMa®-II filter membrane, you can also discard the valve set or alternatively store it for replacement.