SaHoMa[®]
Safety Horse Mask

VETneb[®]
mesh nebulizer

REINIGUNGSANLEITUNG CLEANING INSTRUCTIONS INSTRUCTIONS DE NETTOYAGE

SaHoMa[®]-II Mesh-Verneblereinheit
SaHoMa[®]-II mesh nebulization unit
SaHoMa[®]-II unité de nébulisation

Art.-No.: HM-236

M-neb[®] vet Mesh-Verneblereinheit
M-neb[®] vet mesh nebulization unit
M-neb[®] vet unité de nébulisation

Art.-No.: 20400208

Liebe Kunden,

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Dear customer,

The mesh nebulization unit is a highly innovative product that should be handled with utmost care and attention – your pet will thank you for it.

Thorough cleaning of the inhalation components, especially the mesh nebulization unit, is therefore important in order to be able to carry out hygienic inhalation sessions at all times, avoid the risk of infection and maximize the service life.

These cleaning instructions show you how to do this. Please also refer to the relevant chapters in the detailed instructions for use.

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Connection cable for connecting the SaHoMa®-II control unit or USB board for connecting the M-neb® vet control unit

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Cleaning the mesh nebulization unit

The mesh nebulization unit **should be cleaned after each inhalation process.**

The following pages describe how to clean the mesh nebulization unit

1. after each inhalation process
2. after using brine inhalation solutions and / or
3. with the concentrated detergent.

Use the sterile water for cleaning after each inhalation process and in case of salt incrustations caused by brine inhalation solutions. Cleaning with the concentrated detergent is only recommended when the performance of the mesh nebulization unit noticeably decreases.

1. Cleaning the mesh nebulization unit after each inhalation process

1. Fill approx. 2 ml of sterile water into the medication chamber of the mesh nebulization unit.
2. Shake the mesh nebulization unit and then empty it.
3. Fill again approx. 1 ml of sterile water into the medication chamber of the mesh nebulization unit.
4. Connect the mesh nebulization unit to the control unit and start it as described in the detailed instructions for use. Now wait until the filled sterile water has been completely nebulized. Then switch off the control unit.
5. Remove the mesh nebulization unit from the control unit and allow the opened mesh nebulization unit to air dry on an absorbent surface. Make sure that no residual liquid can run back onto the mesh membrane.

2. Cleaning the mesh nebulization unit after using brine inhalation solutions

Especially when brine inhalation solutions are used, the mesh nebulization unit may be clogged by salt crystals.

Therefore, proceed as follows when cleaning:

1. Fill approx. 5 ml of sterile water into the medication chamber of the mesh nebulization unit.
2. Set the mesh nebulization unit aside for a few minutes and let the sterile water act on the mesh membrane. During this time it dissolves the salt particles.
3. Empty the mesh nebulization unit completely and refill approx. 1 ml of sterile water.
4. Connect the mesh nebulization unit to the control unit and start it as described in the detailed instructions for use. Now wait until the filled sterile water has been completely nebulized. Then switch off the control unit.
5. Remove the mesh nebulization unit from the control unit and allow the opened mesh nebulization unit to air dry on an absorbent surface. Make sure that no residual liquid can run back onto the mesh membrane.

3. Cleaning the mesh nebulization unit with the concentrated detergent

As soon as the performance of the mesh nebulization unit noticeably decreases, cleaning with the cleaning concentrate is recommended. If the mesh nebulization unit is used exclusively with isotonic saline solution (NaCl 0.9 %), cleaning with the concentrated detergent is not necessary. If you notice a drop in performance in this case, this is due to regular wear and tear, which will result in the mesh nebulization unit needing to be replaced in the near future.

Proceed as follows for cleaning with the concentrated detergent:

1. Fill 2-3 ml of the concentrated detergent into the medication chamber of the mesh nebulization unit.
2. Connect the mesh nebulization unit to the control unit and start it as described in the detailed instructions for use. Nebulize a portion of the filled concentrated detergent for approx. one minute. Then switch off the control unit.
3. Now wait approx. 3 minutes so that the concentrated detergent can act on the membrane of the mesh nebulization unit.
4. Then nebulize the remaining concentrated detergent completely as described in step 2.
5. Fill approx. 3 ml of sterile water into the medication chamber of the mesh nebulization unit. Shake the mesh nebulization unit and then empty it.
6. Fill again approx. 1 ml of sterile water into the medication chamber of the mesh nebulization unit and nebulize it as described in step 2.
7. Remove the mesh nebulization unit from the control unit and allow the opened mesh nebulization unit to air dry on an absorbent surface. Make sure that no residual liquid can run back onto the mesh membrane.

Attention

Please make sure that the cable with plug or the USB board on the mesh nebulization unit does not get wet (danger of short circuit)! Never pull directly on the cable or the USB board as this can cause cable breaks and device failures!

Warning

Always ensure optimum ventilation when nebulizing the concentrated detergent! Although the concentrated detergent is not harmful, it should not be inhaled!

Caution

Only use the concentrated detergent offered by the manufacturer for cleaning. It is strongly recommended not to clean the front opening of the mesh nebulization unit by using any object (paper towel, cotton swab, etc.) or finger. This can cause irreparable damage to the membrane of the mesh nebulization unit, so it's no longer working.

Note

Instead of using sterile water, the mesh nebulization units can also be cleaned with distilled water. While sterile water is purest water without microorganisms, such as germs and bacteria, or trace elements, distilled water has only been freed from trace elements and other impurities. Distilled water for medical purposes is also suitable for cleaning the mesh nebulization unit.

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