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## For a quick start ...

ALWAYS ON HAND:  
Made of unbreakable  
plastic film for storage  
in carrying box

### Step 1

Put a tight-fitting halter on your horse and place the sealing ring on the bridge of its nose. Now attach the halter strap so that the transverse strap passes over the bridge of the nose and hook the carabiners on the right and left of the halter. Now slide the housing on over the sealing ring and click the halter strap into place on the housing using the quick-release fasteners.

### Step 2

Insert a filter membrane into the filter basket and attach it to the housing. Now slide the control unit over the guide rails until it clicks into place.

### Step 3

Now fill the mesh nebulization unit with the prepared inhalation solution and attach it by connecting the cable to the right or left output of the control unit. Then slide the cover along the guide rails until it clicks into place.

### Step 4

Start the control unit by pressing the button for approx. 2 seconds via the opening on the side of the protection cover. After the inhalation process, the control unit switches off automatically.

### Step 5

In the end, the used components must be cleaned and dried before being stored.

## Caution

The brief instructions for the SaHoMa®-II inhalation set for horses do not replace the detailed instructions for use in any case. Knowledge and observance of the instructions for use is assumed. In the instructions for use you will find, among other things, further instructions as well as information on changing programs, charging the SaHoMa®-II control unit and cleaning the individual components.