# **Attachment instructions**

We explain below how to correctly attach the SaHoMa®-II fastening strap.

#### The SaHoMa<sup>®</sup>-II fastening strap at a glance Art. no.: HM-217



The SaHoMa®-II fastening strap is a supplement to the SaHoMa®-II halter strap included in the SaHoMa®-II inhalation set for horses. By attaching it behind the horse's head, the SaHoMa®-II fastening strap ensures a much tighter fit. Especially for horses that inhale while moving, this may be necessary under certain circumstances.



NTI\_HM-200\_IFU\_HM-217\_EN\_A | 01.09.2023

# **Attachment instructions**

ANY QUESTIONS ? We are happy to help you: +49 (0) 60 22-610 62-0



### Step 1

Set the SaHoMa<sup>®</sup>-II fastening strap as wide as possible and connect it to one side of the SaHoMa<sup>®</sup>-II housing. To do this, fix a quick-release fastener in the corresponding receptacle on the SaHoMa<sup>®</sup>-II housing. Optionally, you can also attach the SaHoMa<sup>®</sup>-II fastening strap to both sides of the SaHoMa<sup>®</sup>-II housing.

Always make sure that the side adjustment is on the left side (as seen from the horse) and that the chin strap runs under the horse's head to ensure an optimal fit of the SaHoMa®-II fastening strap later on.



### Step 2

After attaching the appropriate SaHoMa<sup>®</sup>-II sealing ring, put the SaHoMa<sup>®</sup>-II housing on the horse. Then put the SaHoMa<sup>®</sup>-II fastening strap on the horse's head behind the ears (and the halter if necessary). If you have only fixed one quick-release fastener to the SaHoMa<sup>®</sup>-II housing so far, you can now also attach the second quick-release fastener to the receptacle on the SaHoMa<sup>®</sup>-II housing.



#### Step 3

Now adjust the SaHoMa<sup>®</sup>-II fastening strap to the horse's head using the side adjustment, which should be under the left eye. Then fix the chin strap, which runs under the horse's head, using the clamp. The chin strap ensures that the SaHoMa<sup>®</sup>-II fastening strap moves away from the horse's eye. Afterwards, the SaHoMa<sup>®</sup>-II housing should be firmly placed on the horse's head. Now attach all other components of the SaHoMa<sup>®</sup>-II inhalation set for horses before starting the inhalation.



### Cleaning the fastening strap

You can remove light dirt from the SaHoMa<sup>®</sup>-II fastening strap by using warm water and a small hand brush. You can also wash the SaHoMa<sup>®</sup>-II fastening strap in the washing machine if it is very dirty. To do this, place the SaHoMa<sup>®</sup>-II fastening strap in a padded washing net and set the washing machine to the gentle cycle at a temperature of 30° C. Use a mild detergent and do not use fabric softener.

